

Ceiling Hoist Help Guide




**Changing Places
Transforming Lives**

proudly supported by  **CHS Healthcare**

1 The Ceiling Hoist - what to do and how it works




- Hold and use the spreader bar to pull the hoist into a clear area for safe use. Make sure there are no obstructions around you
- Take care to familiarise yourself with the features of this hoist

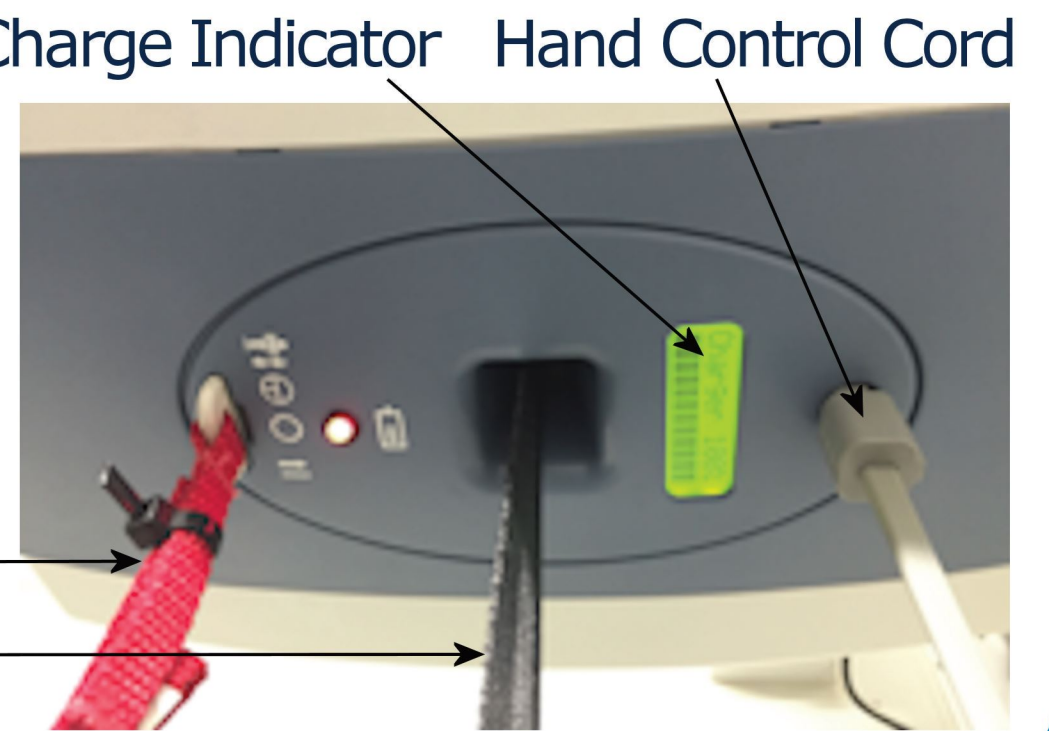

Emergency Stop
Lifting Strap
Spreader bar

2 The hand control, spreader bar and your sling

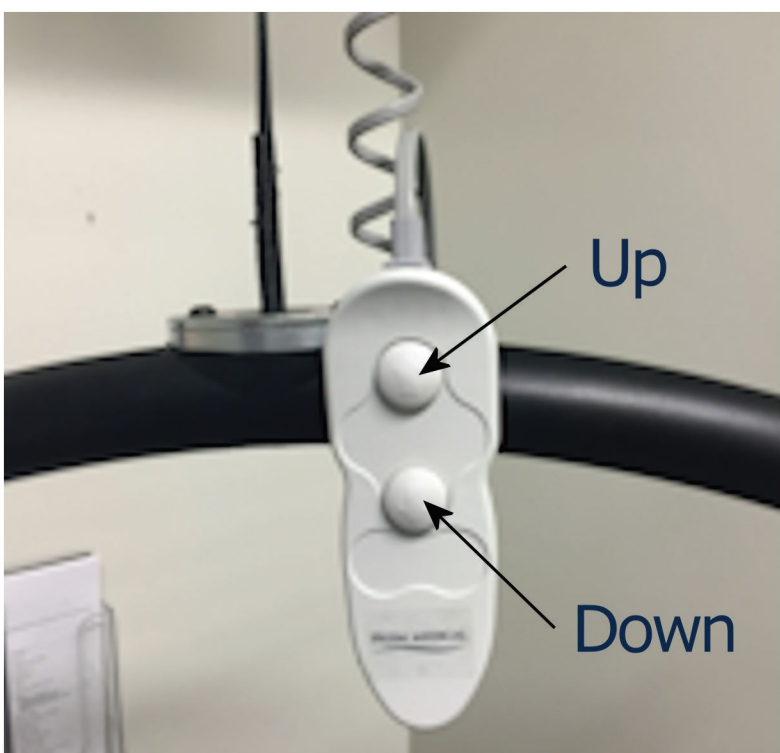


- The hoist has 2 buttons - up and down
- Do not pull hard on the hand control cord
- When the hand control is not in use clip the hand control over the spreader bar (left) to keep it safely out of the way
- Keep all obstructions clear when lifting



Loop style slings only

3 If the hoist doesn't work

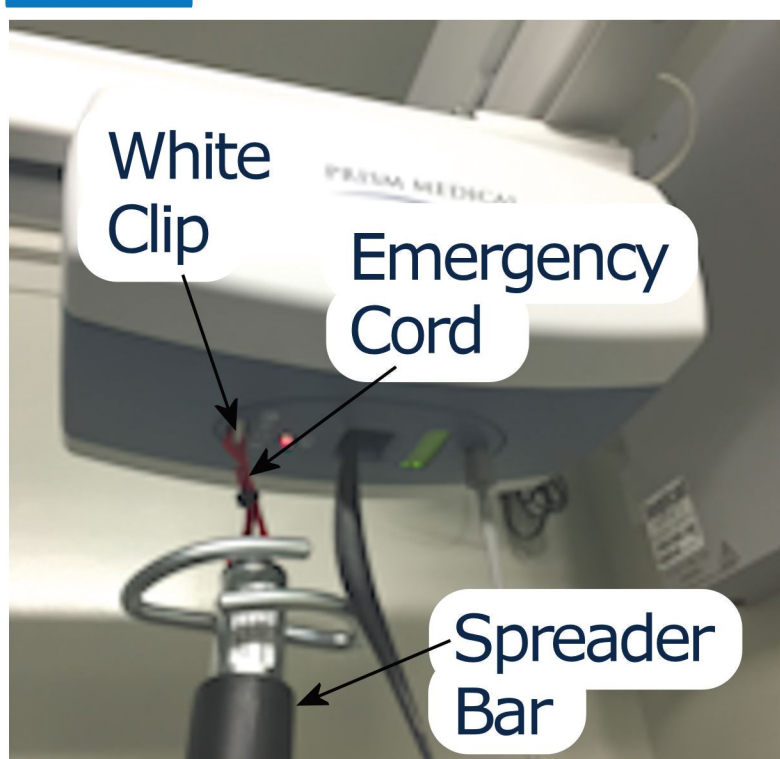

- If the hoist is not working, check to see if the red **Emergency Stop** cord has been pulled into the down position, disabling the hoist.
- **PULL AND HOLD EMERGENCY CORD TO LOWER HOIST IF REQUIRED**
- To re-enable hoist, push white clip attached to the top of the red **Emergency Stop** cord with your finger or use the end of the spreader bar to reach.

4 When you are finished...



Please always return the hoist to the charge position as shown, ready for re-use.